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Safety

CAUTION Risk of injury!

- Follow the accident prevention regulations for the tools and machines used.
- Pay attention to the safety of your fingers when drilling.
- Always drill orthogonally to the surface.

Assembly

Fig. 5 (for pelvic belt) and Fig. 6 (for chest belt) show the device with existing assembly holes.

For devices without existing assembly holes:

Tools: Battery powered drill; Drill bit: Twist drill Ø 8 mm

Pelvic belt: Using the drilling jig, according to Fig. 1, drill a hole through the outside wall only.

Chest belt: Using the drilling jig, according to Fig. 3, drill a hole through the plastic rib.

For all devices:

(Pelvic belt: Fig. 2 & 5 / Chest belt: Fig. 4 & 6)

Tools: Allen key SW4

1. Lead bolt Ⓑ with spacer Ⓒ through the eyelet Ⓐ of the belt. If the belt ends protrude, fold them and lead the bolt through two eyelets.
2. From the inside (for the pelvic belt) resp. from the frontside (for the chest belt), place the rivet nut Ⓓ in the drilled hole.
3. Tighten the screw with the Allen key until the rivet nut completely fits with its bearing face.
4. Repeat steps 1 to 3 on the opposite side.

© 2021, Invacare GmbH, D-88316 Isny, all rights reserved.
Part No. 1535808-B / 2021-10

AQUATEC®

Yes, you can!