


Pure Bay (1)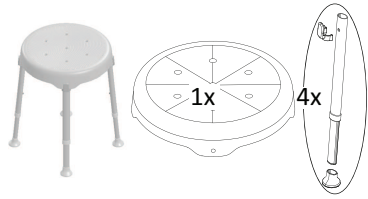


Pure Sol (2)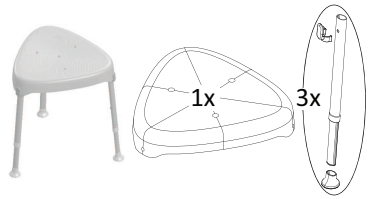


Pure Siro (3)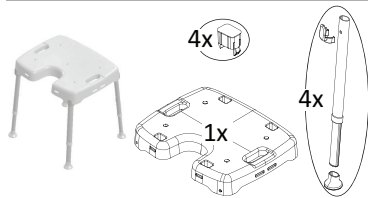


Pure Isla (4)

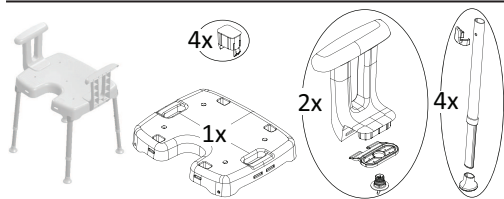
Pure Isla (5)

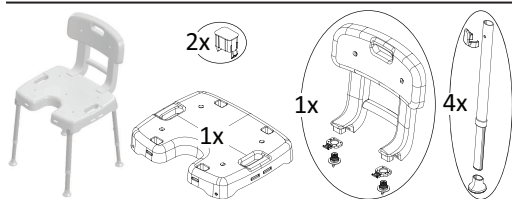Pure Isla (6)

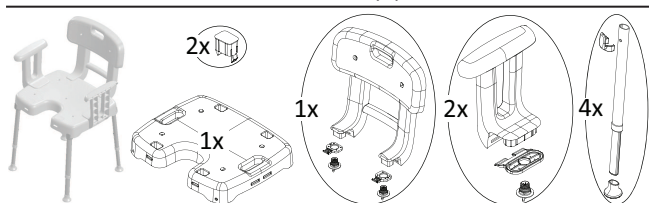Pure Isla (7)

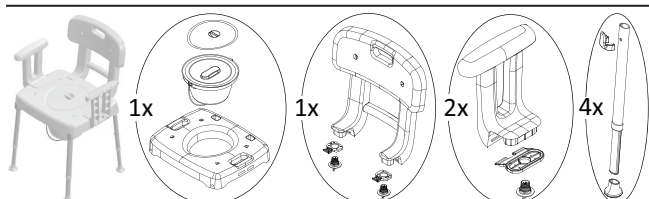Pure Azure (8)

Aquatec® Pure Bay/Pure Sol/ Pure Siro/Pure Isla/Pure Azure

en Shower Stool / Shower- and Toilet Chair

User Manual 4

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
lv Dušas taburete / dušas un tualetes krēsls

Instrukciju rokasgrāmata 32

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

This manual MUST be given to the user of the product. BEFORE using the product, this manual MUST be read and saved for future reference.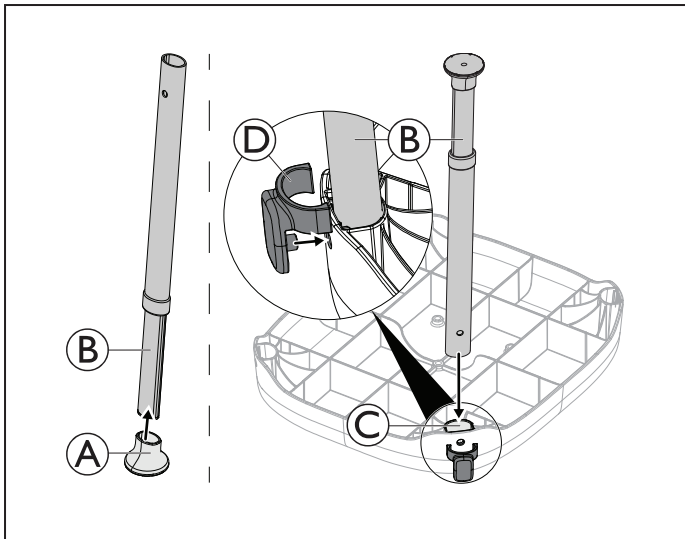


(9)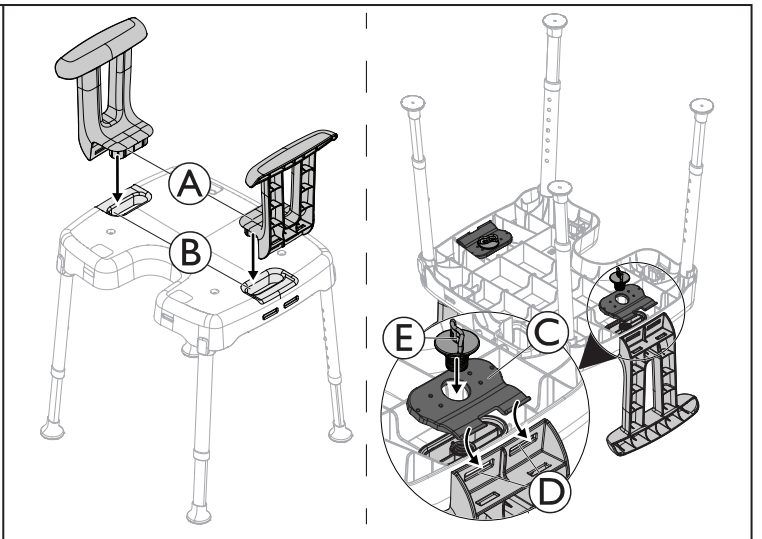


(10)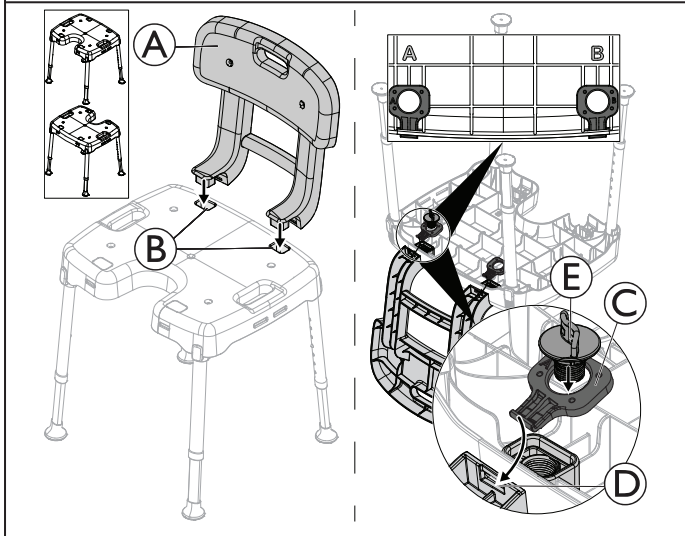


(11)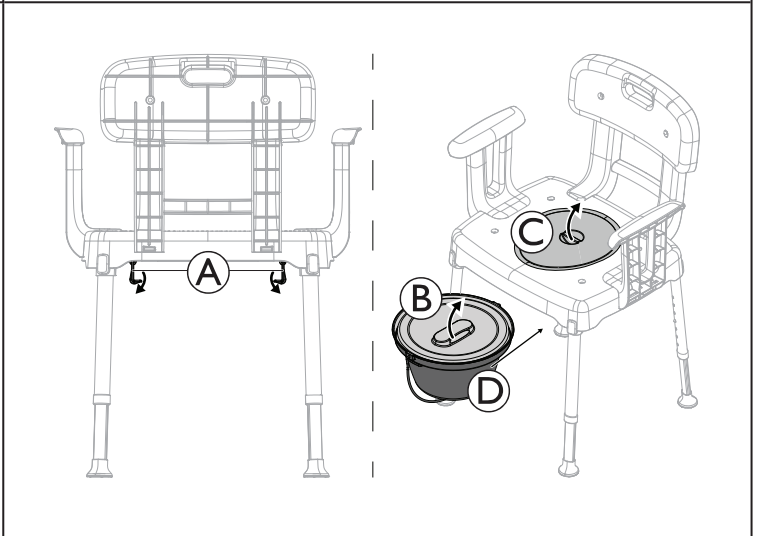


(12)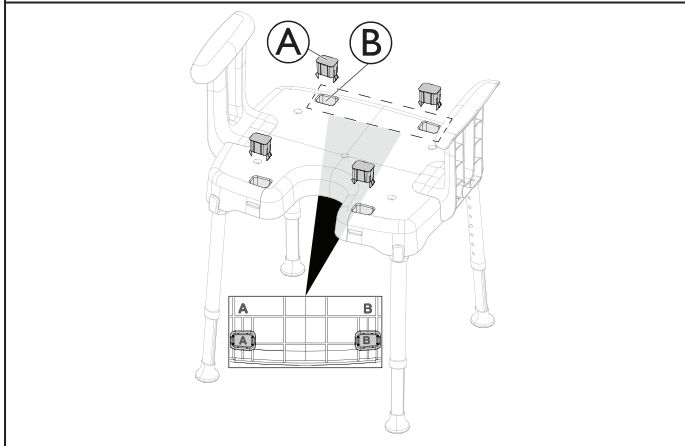


(13)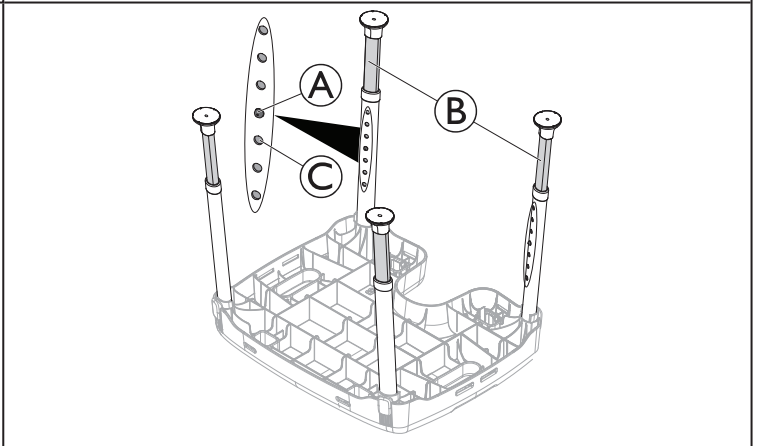


(14)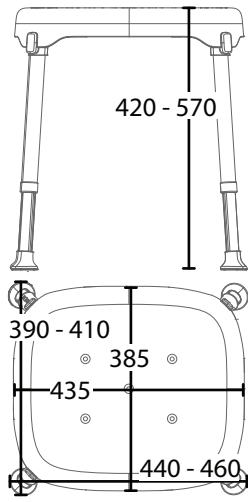


(15)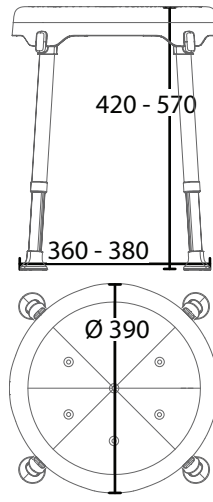


(16)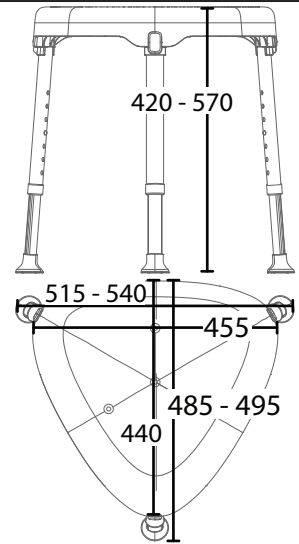


(17)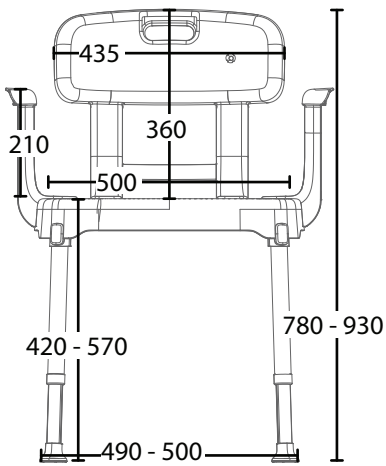


(18)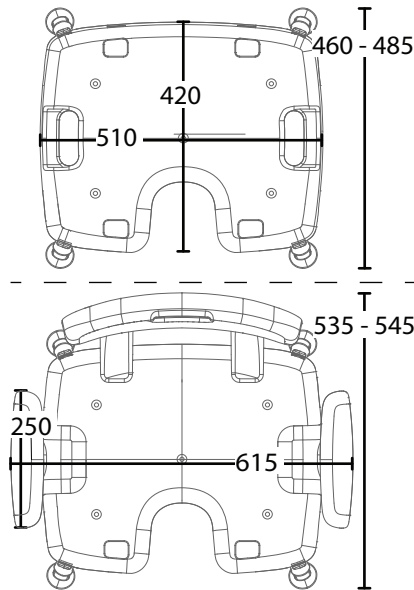

(19)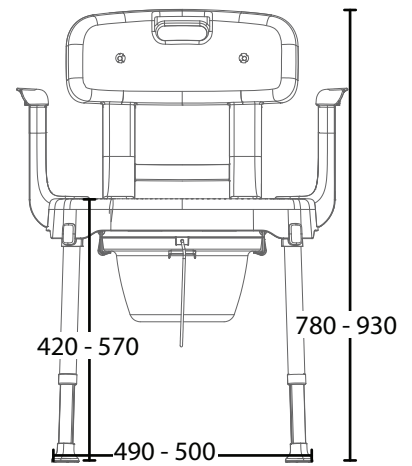

(20)

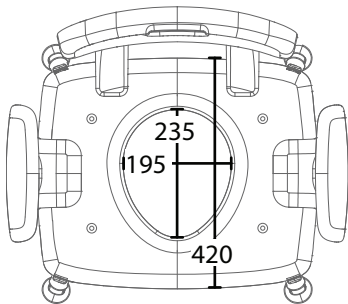(21)

aquatec

Pure XXXXX

REF (A) dd/mm/yy

LOT (E) MD (H)

XXX kg

MADE IN ITALY

UK CA CE J

UDI (01) (11) (10)

(D) (C) (B) (F) (G)

(22)

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website. If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website.

Invacare reserves the right to alter product specifications without further notice.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

1.2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

1.3 Compliance

This CE- and UKCA-marked medical device conforms to: EU Medical Device Regulation, UK Medical Device Regulation, ISO 13485, ISO 14971, REACH, ISO 17966 and related standards.

1.4 Service Life

The expected service life of this product is five years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

1.5 Limitation of Liability

Invacare accepts no liability for damage arising from non-compliance with the user manual, incorrect use, natural wear and tear, incorrect assembly or set-up, unauthorised modifications, combinations and/or use of unsuitable spare parts.

2 Safety

2.1 General Safety Information

WARNING!

Risk of serious injury or damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.

- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material.

WARNING!

Risk of falling

- Do not use if defective.
- Do not make any unauthorised alterations or modifications to the product.
- Do not move or displace the product under load or during use.
- Do not stand on the product.
- Do not use or store the product outdoors.
- Do not tilt.
- Do not sit on the product if not all legs have full contact to the level ground.

WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

CAUTION!

Risk of injury

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalogue or website for available options or contact your Invacare distributor.

2.2 Labels and Symbols on the Product

Identification Label (Figure 22)

(A)	Reference number	(F)	Maximum user weight
(B)	Manufacturer	(G)	Made in Italy
(C)	Production date	(H)	Medical Device
(D)	Unique Device Identifier	(I)	Read user manual
(E)	Lot number	(J)	European Conformity UK Conformity Assessed

The identification label is attached to the bottom side of the seat plate.

3 Product Overview

3.1 Intended Use

The Aquatec Pure Bay, Pure Sol and Pure Siro are shower stools in different shapes.

The Aquatec Pure Isla is a modular shower chair with a hygiene recess.

The Aquatec Pure Azure is a modular shower and toilet chair with a toilet opening that can be used over standard toilets when the toilet pan is removed.

Both Pure Isla and Pure Azure have removable armrests and backrests. All models have height-adjustable legs and anti-slip feet.

All devices are intended to be used as a shower aid and Pure Azure is additionally used as toilet aid.

Intended Users

Users are any adults and adolescents with restricted or limited mobility or limited muscle strength. If used without an assistant, the users must be able to keep their own balance. The recommended body height for users is between 146 and 200 cm.

Indications

- Restricted or limited mobility (e.g. legs and/or hips) or
- having balance issues to stand or walk due to transitional or permanent disabilities.

Contraindications

There are no contraindications known when using the product as intended. For the maximum user weight, see *7.1 Weights and Materials*, 6.

3.2 Scope of Delivery

The scope of delivery depends on the version ordered. The possible versions are shown in figures 1 to 8.

4 Setup

CAUTION!

Risk of pinch points

- Take care during assembly.

Installing the Legs (Figure 9)

1. Install the feet **A** by pressing them completely onto the legs **B**.
2. Place the seat plate top side down on an even surface and install the legs **B** to the slots **C** in the seat plate.
3. Secure the legs **B** with the clips **D**.

Installing the Armrests - optional (Figure 10)

1. Insert the armrests **A** from above into the cut outs **B** in the seat plate.
2. Turn the product upside down and slide the armrest fixation plates **C** into the fixation slots **D**.
3. Fix the armrest fixation plates **C** to the seat plate by using the hand screws **E**.

Installing the Backrest - optional (Figure 11)

The hygiene cut out in the seat plate of Aquatec Pure Isla can face to the rear or to the front.

1. Insert the backrest **A** from above into the cut outs **B** in the seat plate.
2. Turn the product upside down and slide the backrest fixation plates **C** into the fixation slots **D**.
 -  Refer to A+B markings on the bottom side of the seat plate and backrest fixation plates for correct orientation.
3. Fix the backrest fixation plates **C** to the seat plate by using the hand screws **E**.

Installing the Toilet Pan - optional (Figure 12)

1. Fold down toilet pan holders **A** on the bottom side of the seat plate.
2. Remove toilet pan lid **B** from the toilet pan and toilet cover lid **C** from the seat plate.
3. Insert the toilet pan **D** onto toilet pan holders **A** from front or rear.

Installing the Cover Plugs - optional (Figure 13)

CAUTION!

Risk of pinch points

- Do not use the product without cover plugs.

1. Insert the cover plugs **A** from above into the cut outs **B** in the seat plate.

Refer to A+B markings on bottom side of the seat plate and cover plugs for correct orientation.

Adjusting the Seat Height (Figure 14)

1. Press snap button **A** on the legs and slide the leg extension **B** up or down to the desired adjustment hole **C**.

All legs generally should be adjusted to the same height. **Only for Pure Isla and Pure Azure:** As an exception, the front legs may be lowered by one position to improve the seating comfort when restricted posture is given due to problems with the coccyx etc.

Disassembly

The product is disassembled in reverse order to assembly.

5 Maintenance

5.1 General Maintenance Information

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

5.2 Cleaning and Disinfection

General Safety Information

CAUTION!

Risk of contamination

- Take precautions for yourself and use appropriate protective equipment.

NOTICE!

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
- Always make sure that the product is completely dried before taking into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

5.2.1 Cleaning Intervals

! NOTICE!
Regular cleaning and disinfection enhance smooth operation, increases the service life and prevents contamination.
Clean and disinfect the product:

- regularly while in use,
- before and after any service procedure,
- when it has been in contact with any body fluids,

5.2.2 Cleaning Instructions

! NOTICE!
Maximum cleaning temperature:

- 60 °C by hand
- 85 °C for 3 minutes by automatic washing systems

1. Clean the product by wiping down all generally accessible surfaces.
2. Dry the parts with a cloth.

5.2.3 Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

6 After Use

6.1 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see *5 Maintenance, 5*.

Make sure that the user manual is handed over with the product.

6.2 Disposal

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

7 Technical Data

7.1 Weights and Materials

The dimensions of the products are shown in figures 15 to 21. The values shown in the figures are millimeters.

Maximum User Weight	
Pure Bay	180 kg

Maximum User Weight	
Pure Sol	150 kg
Pure Siro	130 kg
Pure Isla	160 kg (150 kg*)
* Item no. 1655998	
Pure Azure	160 kg

Product Weight	
Pure Bay	2.3 kg
Pure Sol	2.1 kg
Pure Siro	2.2 kg
Pure Isla	2.8 kg
Pure Isla with armrests	3.5 kg
Pure Isla with backrest	3.7 kg
Pure Isla with armrests and backrest	4.4 kg
Pure Azure	5.1 kg

Materials	
Seat / armrests / backrest / lid / toilet pan and lid	Polypropylene
Legs	Aluminium
Feet	TPE

All product components are corrosion resistant and do not contain natural rubber latex.

7.2 Environmental Parameters

Operating Conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Storage and Shipping Conditions	
Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing

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Invacare AG
Invacare Limited
Invacare Poirier SAS
Invacare GmbH
Invacare UK Operations Limited